

SBRT SOLUTION ARM REST PLATE FOR UPPER ARM SUPPORTS AND WRIST SUPPORT



Article No. : 32317/15

A. GENERAL PRODUCT INFORMATION

The products referred to in these instructions are medical devices, used for patient positioning and immobilisation in radiation therapy.

This product may only be used in combination with Orfit immobilisation products. Orfit prohibits the use of unauthorised third-party products in conjunction with its own products.

B. PRODUCT DESCRIPTION


This arm rest plate is attached to the SBRT BASE PLATE and functions as an indexation device for the ARM REST CUSHIONS and GRIP POLES. Information on these other parts can be found in the respective 'instructions for use' and on www.orfit.com.

C. INSTALLATION

1. The arm rest plate is attached to the pins on the cranial section of the SBRT (long or short) base plate.



2. The arm rest is foreseen with indexing features to be able to use the adjustable arm supports, wrist supports and hand grips as shown on below picture.

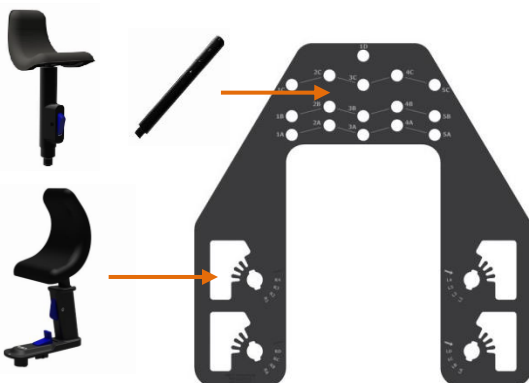

D. STORAGE

Always store the product in a safe place to prevent it from getting damaged. Store the frame between +10°C (50°F) and 40°C (122°F).

E. PROPERTIES


E.1. Physical Properties

Dimensions: 550 mm L x 540 mm W x 27,5 mm H
 Weight: 2.68 kg
 Material: High Pressure Laminate

This product is made entirely of electrically non-conductive, non-metallic and non-magnetic materials and is MRI safe.



E.2. Mechanical Properties

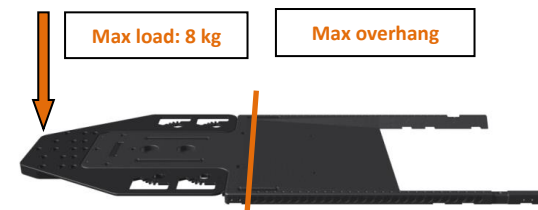

The arm rest plate adds 162 mm (6,38 inches) to the total length of the SBRT system which then is 2435 mm long (7,31 feet).

F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

The product can be disposed of with household waste.

G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.



ORFIT INDUSTRIES
 Vosveld 9A | B-2110 Wijnegem | Belgium
 T (+32) (0)3 326 20 26
welcome@orfit.com

www.orfit.com



Ref. No. 50084
 VERSION 3
 LAST UPDATE: 02/04/2021
 REVISION DATE: 02/04/2023